



DENTIN

Temporary filling material

COMPOSITION

Vaseline oil, Lanoline, Poli-dimethyl-siloxane, Zinc oxide, Zinc sulfate, Calcium sulfate, Caoline, Pigment

PURPOSE

Dentin does not contain eugenol, it is used in therapeutic dentistry for temporary filling and for covering drugs in the tooth cavity.

PROPERTIES

Dentin is a paste made on the basis of zinc sulfate and mineral oil. It is uniform paste of white or beige color without external additions of dry places, initial setting time: not more than two hours, final setting time: not more than 24 hours, density: 2.8-3.2 g/cm³.

CLINICAL PROCEDURE

Insert Dentin into the prepared cavity, in a thin layer of 1-2 mm, by spatula. The first stage of hardening in the mouth takes 2-3 hours, it is not recommended to the patient to eat during this time. The final hardening takes 4-8 hours.

The paste is removed with excavator or dental probe. Using ultrasonic probe is permitted.

Tightly close the jar with paste after filling in order to prevent moisturizing.

CONTRAINDICATION

There is no contraindication

PRECAUTIONARY MEASURES

Use in strict accordance with instruction for use

TRANSPORTATION

The material may be transported by all types of covered transport vehicles in accordance with rules of transportation for each type of transport. It may be transported via post, by cars and in containers. Temperature during transportation should be within +(5-24)°C

STORAGE.

The material should be stored at temperature +(5-24)°C in a dry place. Avoid high moisture of air and contact with liquids.

PRODUCER GUARANTEE

The producer guarantees correspondence of material to all technical requirements on condition of observing all rules of utilization, transportation and storage

SHELF LIFE

Shelf life of the material is 2 years. Do not use after expiration of shelf life.

PLACES OF USE

For use in hospitals, dentistry clinics and mobile medical complexes.

DISPOSAL

Disposal of material does not require special precautionary measures

Producer: "Stomadent LLC"

11 Garshina Street, Tomilino, Moscow region, 140070 Russia

Tel. (+7-495) 514-93-47, 514-93-13 Tel/fax. (+7-495) 514-93-46

E-mail: info@stomadent.ru

www.stomadent.ru